


How to wear N95 mask

HOW TO WEAR N95 MASK

1 PUT ON N95 MASK

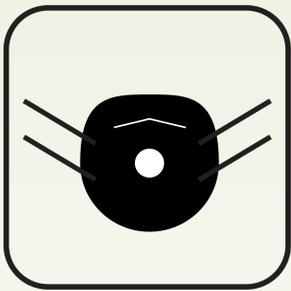

- Select respirator according to fit testing.



- Place over nose, mouth and chin.
- Fit flexible nosepiece over nose bridge.



- Secure on head with top elastic followed by bottom elastic.
- Adjust to fit.

2 PERFORM FIT CHECK



- Inhale – respirator should collapse.
- Exhale – check for leakage around face.


FOR AEROSOL GENERATING PROCEDURES, IN ADDITION TO N95, YOU WILL ALSO NEED



1. Eye protection (face shield or goggles).



2. Gown (clean non-sterile, long-sleeved).

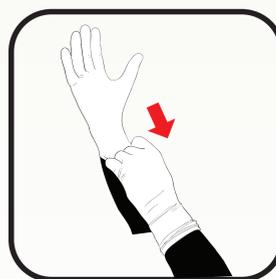

3. Non sterile gloves (some procedures may require sterile gloves).



4. If splashes are expected, put on impermeable apron.